


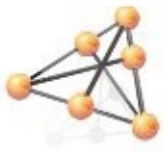

ULTRALINQ[™]
HEALTHCARE SOLUTIONS

Logiq E, I; Vivid E Configuration





1. Start by pressing either the “Config” key on the keyboard.
2. Click on “Connectivity”, and go to the “TCP/IP” tab.

The screenshot shows the configuration interface with the following elements:

- Navigation tabs: TCP/IP (selected), Device, Service, Dataflow, Button, Removable Media, Miscellaneous.
- Computer Name: VIVIDE
- IP settings section:
 - Enable DHCP: (checked)
 - IP-Address: 0.0.0.0
 - Subnet Mask: 0.0.0.0
 - Default Gateway: 0.0.0.0
 - Network Speed: Auto Detect (dropdown menu)
- Footer: Reboot the system to activate any changes saved from this page!


3. If the “Enable DHCP” box is checked, de-select it. This will allow you to specify an IP address for the device.

You will need to reboot the device after changing the IP address.

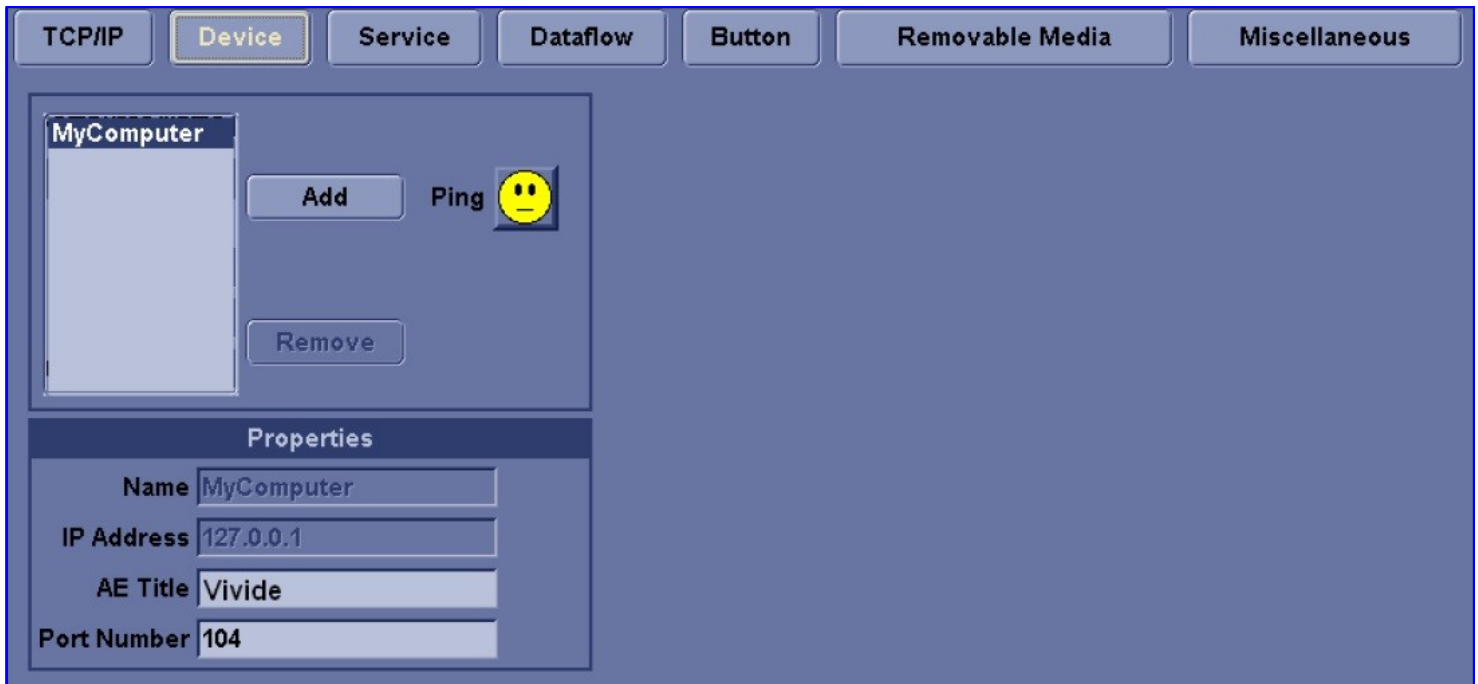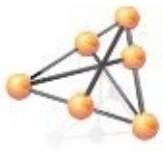

4. Next, click to the “Device” tab. Select “Add” to create a new device. Call the new device “UltraLinq”, and enter the IP address of the computer.

You do not need to adjust the AE Title or Port.

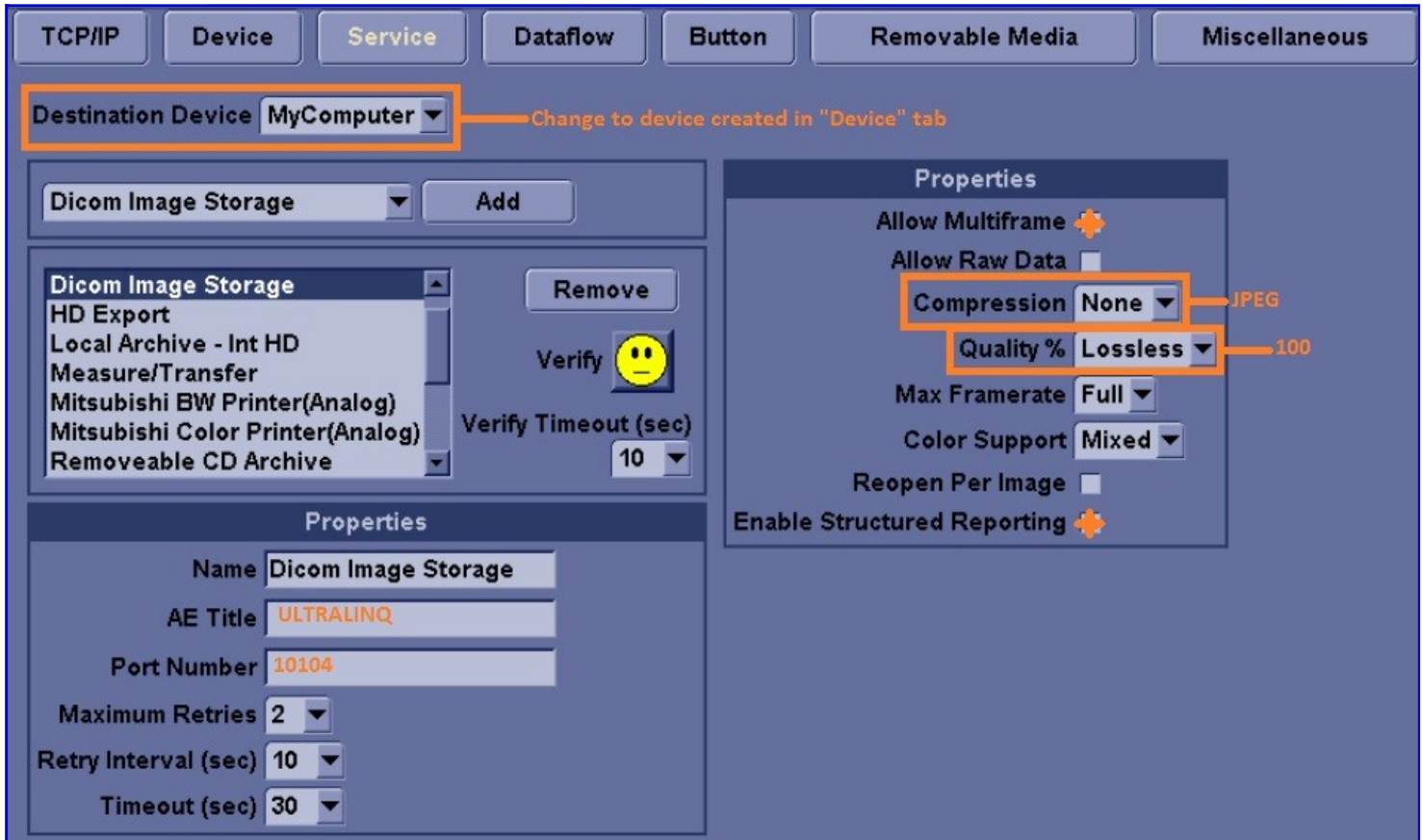


5. The “Service” tab is where you will configure all of the settings for the DICOM server.

6. Begin by changing the selection from the “Destination Device” drop-down to the same device you just added.

7. In the drop-down box immediately underneath that, select “DICOM Image Storage” and hit “Add”. This will make the properties fields appear.

8. Enter the information as shown above.

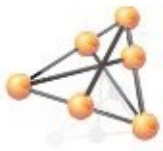

ULTRALINQ™
HEALTHCARE SOLUTIONS
